




cobas Liat System  
*Lab in a tube*

# Roche Molecular Diagnostics

## *A rich history of innovation in PCR & molecular automation*

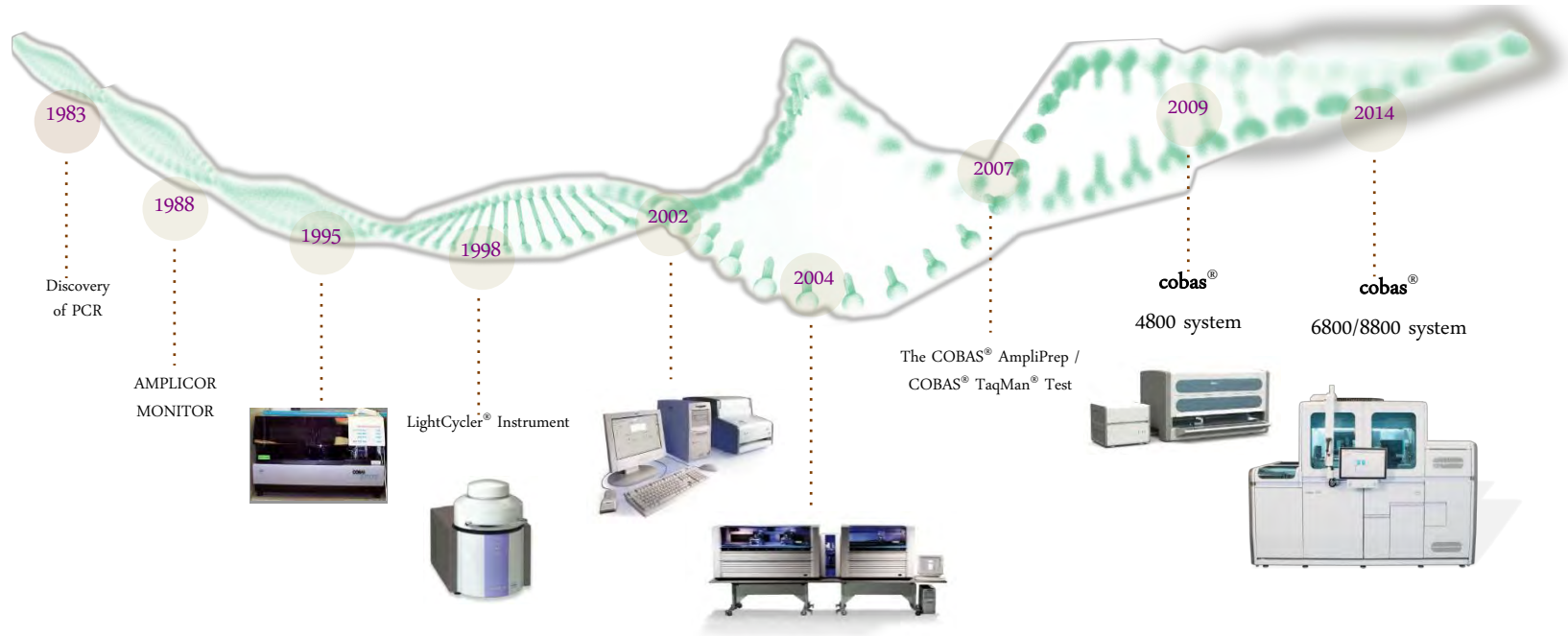


For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 2 | © 2016 Roche



Today, we put a **Lab In A Tube**.

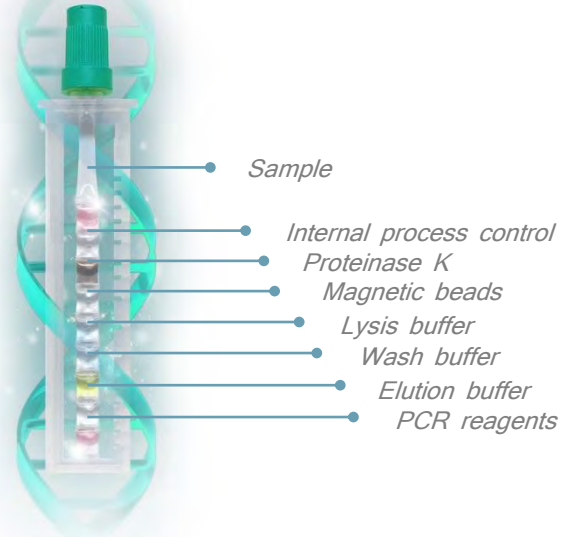

and deliver real-time PCR results **less than 30 minutes\***.



For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 3 | © 2016 Roche

\* Depends on different assays



## Key features

IT connectivity



### ANALYZER



### ASSAY TUBE



Fast Turn Around Time

Ease of Use

Lab Quality Performance

\*cobas® Strep A -15 min; cobas® Influenza A/B, cobas® Influenza A/B & RSV and cobas® Cdiff -20 min; In Development: cobas® MRSA/SA & cobas® HIV-1/2 Qual

For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 4 | © 2016 Roche

# System Overview

## *cobas® Liat® System*



For Roche Internal Use. Do Not distribute.

The **cobas**® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 5 | © 2016 Roche



# cobas<sup>®</sup> Liat<sup>®</sup> System

*A complete solution from sample in to result out*



For Roche Internal Use. Do Not distribute.

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 6 | © 2016 Roche



# cobas<sup>®</sup> Liat<sup>®</sup> System

cobas<sup>®</sup> Liat<sup>®</sup> System

cobas<sup>®</sup> Liat<sup>®</sup> Analyzer



cobas<sup>®</sup> Liat<sup>®</sup> System  
assay tube

Compact, innovative **real-time PCR** platform designed for **on-demand STAT testing**, enabling **confidence** in rapid patient management, at the point of care or in the laboratory.

19.0 cm



11.4 cm



## ANALYZER



## ASSAY TUBE

A pencil-sized, flexible single-use tube acts as the sample vessel and contains all assay reagents pre-packed in tube segments

\*Currently available in the U.S.

For Roche Internal Use. Do Not distribute.

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 7 | © 2016 Roche



# cobas<sup>®</sup> Liat<sup>®</sup> Analyzer




Assay tube entry door

Intuitive touchscreen display



Touchscreen soft keys

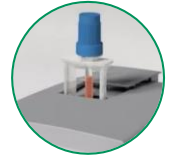

Function buttons

Laser barcode scanner



Navigation buttons

Power on/off



For Roche Internal Use. Do Not distribute.

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 8 | © 2016 Roche






# Assay tube





Pre-packed reagents in a single assay tube



Automated sample preparation & silica bead-based nucleic acid extraction in tube

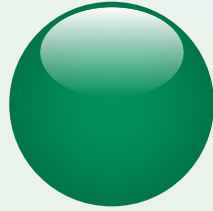

Completely closed system

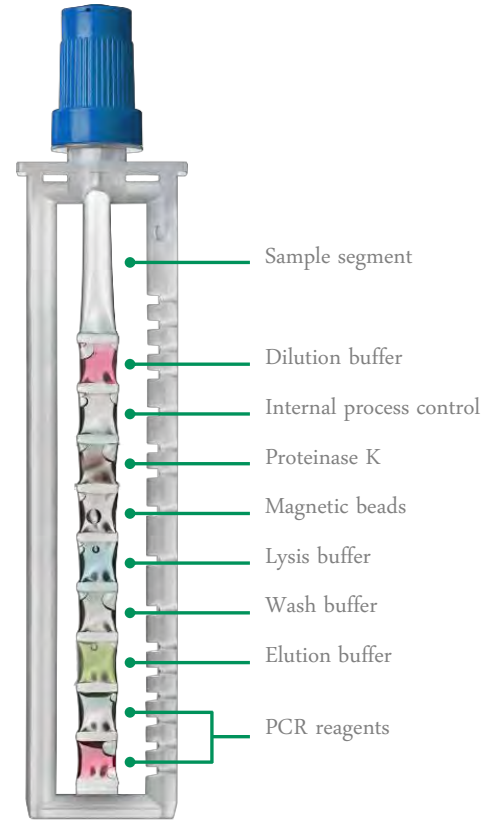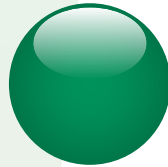

\*Some assay tubes include dilution buffer and/or Uracyl N Glycosylase enzyme. Contents varies by assay and coloring is for illustrative purposes only.

\*Contents varies by assay and coloring is for illustrative purposes only.

**For Roche Internal Use. Do Not distribute.**

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 9 | © 2016 Roche



# Secure and Simple Workflow

Results in  
**30 MINUTES**  
or less\*



1

## Sample

Add patient sample to the **cobas**<sup>®</sup> Liat<sup>®</sup> assay tube with provided transfer pipette



2

## Scan

Scan assay tube using built-in barcode scanner



3

## Start

Insert assay tube into the **cobas**<sup>®</sup> Liat<sup>®</sup> Analyzer

\* Varies by assay

For Roche Internal Use. Do Not distribute.

The **cobas**<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 10 | © 2016 Roche



# cobas® Liat® System

*A complete solution from sample in to result out*



For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 11 | © 2016 Roche



# Software



Assay reagent lot checks & internal control processed with every sample



QUALITY CONTROL

Secured login & keyed user roles for operation and report review



USER SECURITY

Sample process monitoring and “insufficient sample volume” warnings



VOLUME SENSING

Comprehensive self-monitoring and continuous self-calibration



REAL-TIME MONITORING

Auto analysis, interpretation & reporting; no user interpretation required



AUTOMATED DATA INTERPRETATION



For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 12 | © 2016 Roche




# Connectivity

The **cobas**® Liat® Analyzer supports HL-7 protocol and the POCT1-A Standard\*

## Available driver:

**cobas** IT1000

\*Please reference the latest **cobas**® Liat® Analyzer Host Interface Manuals for EDI and DML full technical details

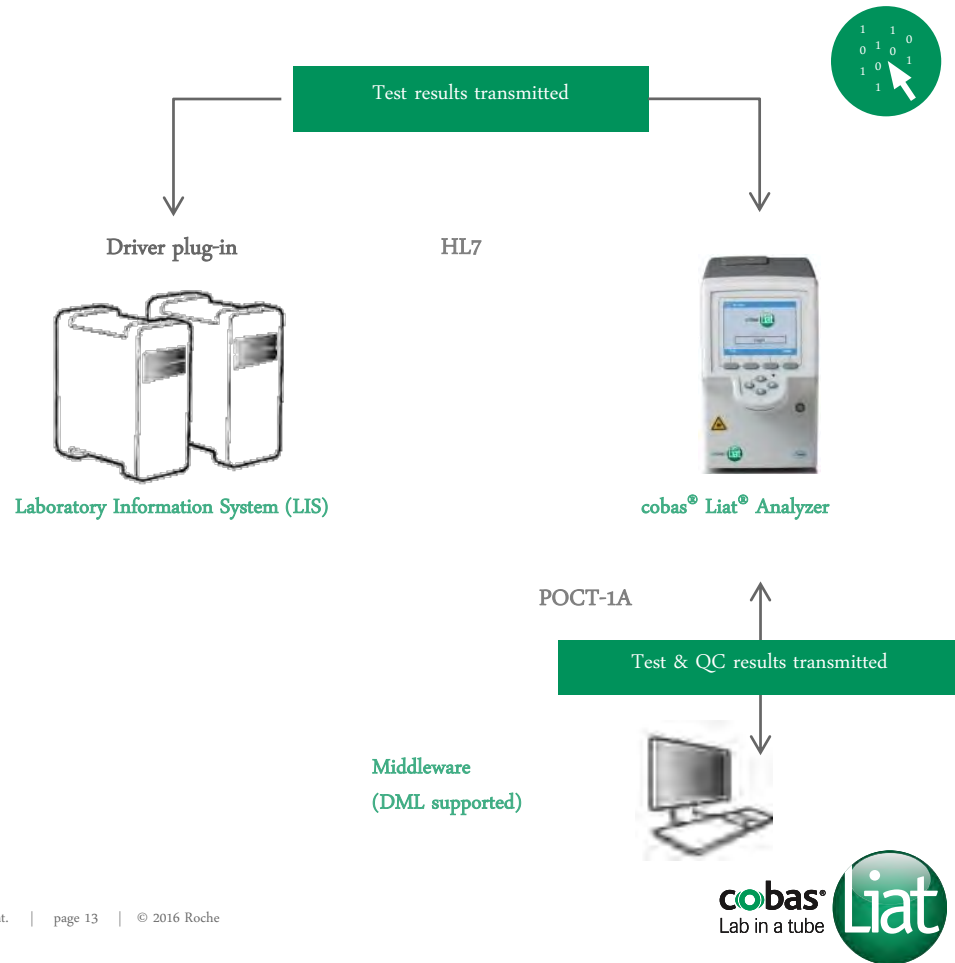

For Roche Internal Use. Do Not distribute.

The **cobas**® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 13 | © 2016 Roche



# cobas<sup>®</sup> Liat<sup>®</sup> System

*A complete solution from sample in to result out*

ASSAY



For Roche Internal Use. Do Not distribute.

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 14 | © 2016 Roche




# Secure and Simple Workflow

Results in  
**30 MINUTES**  
or less\*



Intuitive touchscreen  
user interface

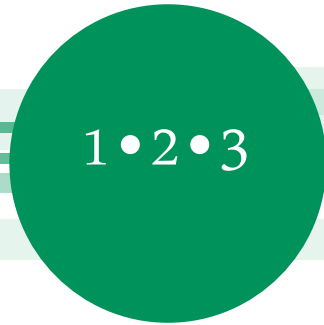

Step-by-step guided instructions



No operator intervention  
or interpretation



No laptops or additional equipment required

\* Varies by assay

For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 15 | © 2016 Roche



# Assay Menu

Respiratory

Influenza  
A/B

Strep A

Influenza  
A/B & RSV



Cdiff

HAI

MRSA/SA




For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 16 | © 2016 Roche






# Assay Menu

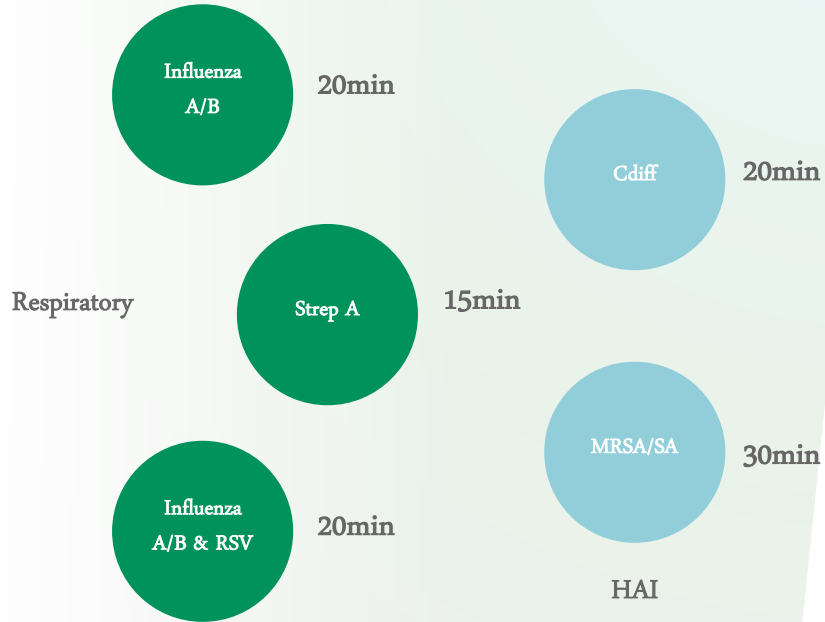

Results in  
**30 MINUTES**  
or less\*

Future pipeline

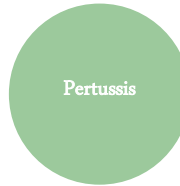

For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 17 | © 2016 Roche






For Roche Internal Use. Do Not distribute.

The **cobas**® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 18 | © 2016 Roche




# Influenza: Evolution in Action

## *Emerging Flu Strains are a Constant Challenge*

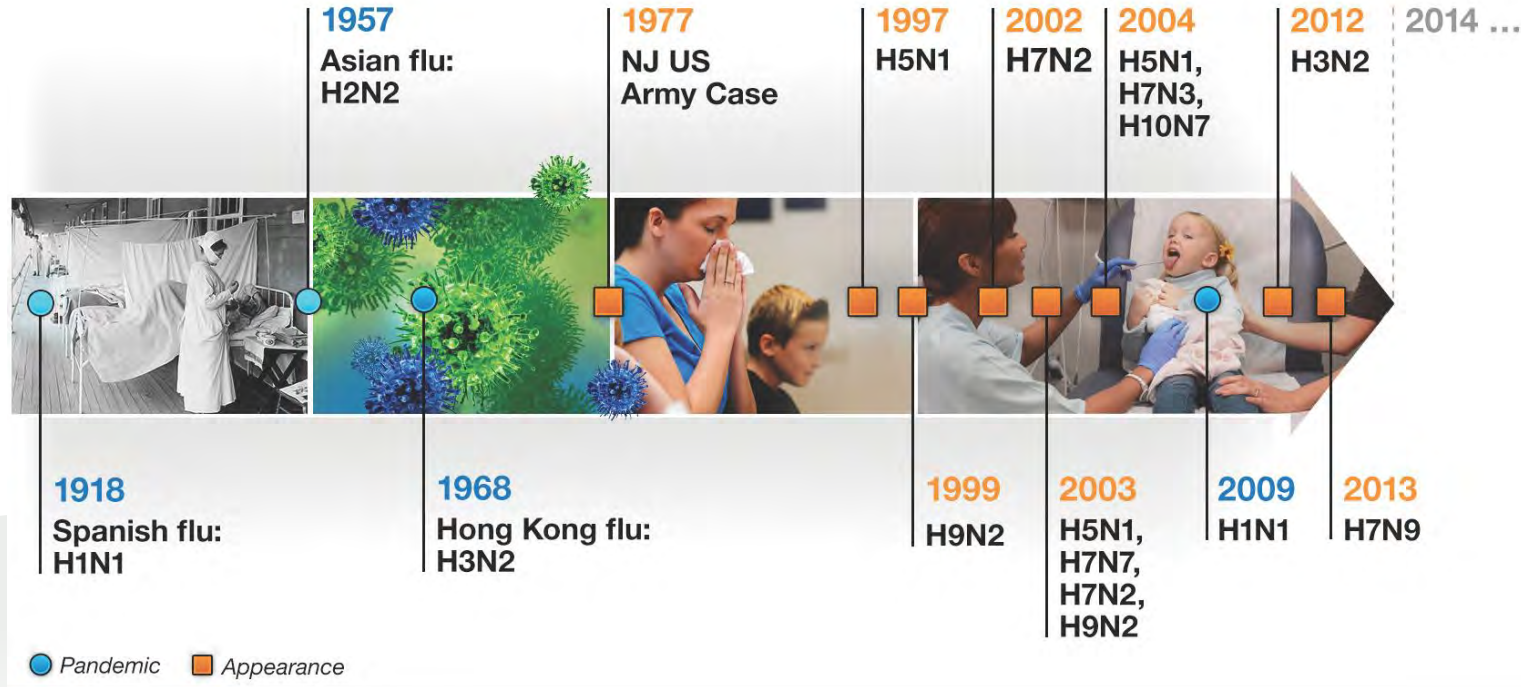

# it a Cold or Flu?

COLD	SYMPTOM	FLU
<input checked="" type="radio"/> YES	STUFFY OR RUNNY NOSE	<input type="radio"/> SOMETIMES
<input checked="" type="radio"/> YES	SNEEZING	<input type="radio"/> SOMETIMES
<input checked="" type="radio"/> YES, with green or yellow gunk	COUGH	<input checked="" type="radio"/> YES, a dry cough
<input checked="" type="radio"/> YES	SORE THROAT	<input type="radio"/> SOMETIMES
<input type="radio"/> MILD head & body aches	BODY ACHEs	<input checked="" type="radio"/> SEVERE aches all over
<input checked="" type="radio"/> NO	NAUSEA	<input type="radio"/> SOMETIMES
<input checked="" type="radio"/> RARE for adults	FEVER	<input checked="" type="radio"/> YES
<input checked="" type="radio"/> NO	CHILLS & SWEATS	<input checked="" type="radio"/> YES
<input type="radio"/> SLOWLY, over a few days	WHEN DO THE SYMPTOMS COME ON?	<input checked="" type="radio"/> FAST, within hours


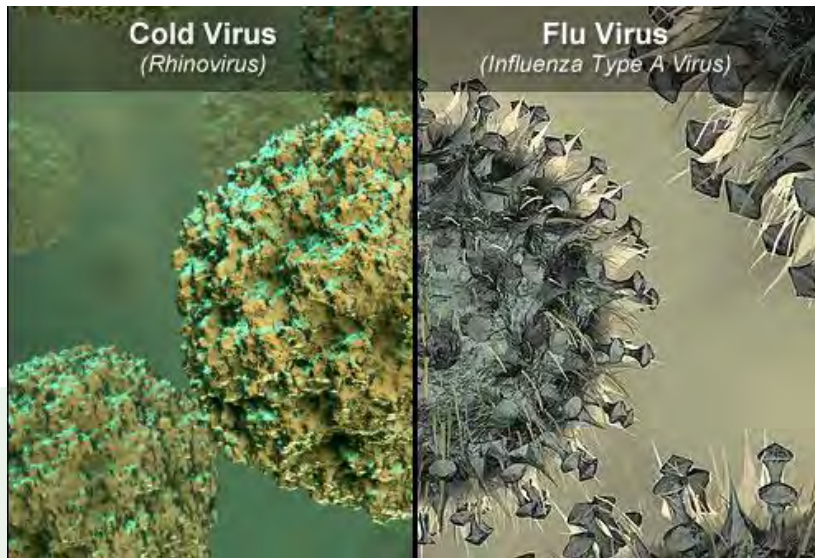


For Ro  
The co



## Aid in diagnosis

If non-pattern in geography or in season, diagnostic tool is required

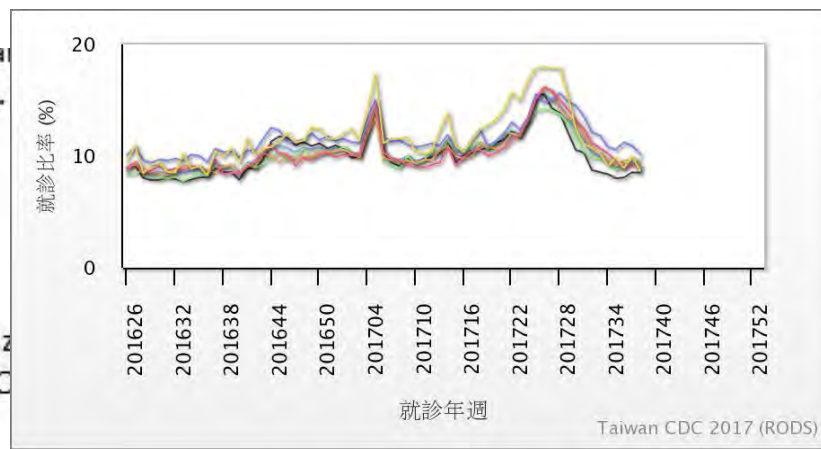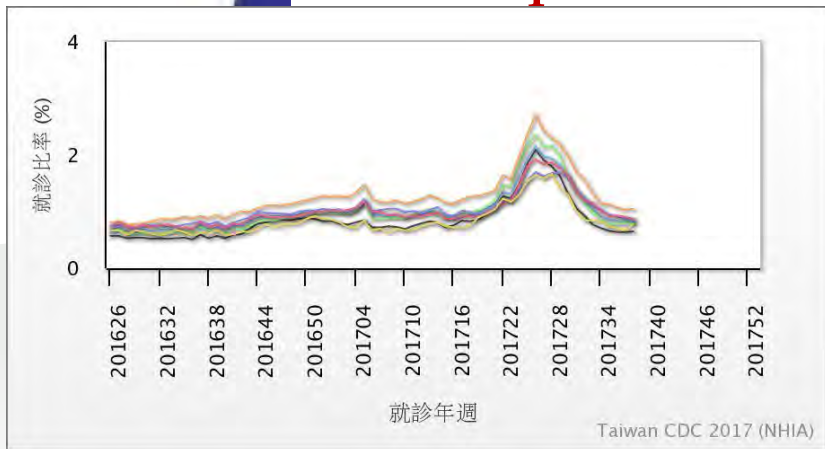



Influenza Test	Advantages	Disadvantages	
<b>Rapid influenza diagnostic tests</b>  Antigen detection (EIA)  Neuraminidase detection assay	Fast (10-30 min)  High specificity	Low-to-moderate sensitivity  False negative results  No subtyping	Point-of-Care Tests
<b>RT-PCR</b>  Conventional gel-based PCR  Real-time RT-PCR  Multiplex PCR	Fast (20 min-6 h)  High sensitivity  Very high specificity	Expensive  Trained technologist	Point-of-Care Tests  Laboratory-Based Tests
<b>Immunofluorescence</b>  Direct fluorescent antibody  Indirect fluorescent antibody staining	Moderately high sensitivity  High specificity	Moderate (2–4 h)	Laboratory-Based Tests
<b>Viral culture</b>  Shell vial culture  Isolation in cell culture	Moderately high sensitivity  Highest specificity	Slow (1–14 days)	Laboratory-Based Tests  (for public health surveillance)



# cobas<sup>®</sup> Influenza A/B

## Key specifications

Instrument	cobas <sup>®</sup> Liat <sup>®</sup> Analyzer
Viral RNA coverage	Influenza type A   Influenza type B
Collection device	Nasopharyngeal Swabs
Media Type	Universal Transport Media (UTM)
Test kit configuration	20 cobas <sup>®</sup> Liat <sup>®</sup> assay Tubes
Reagent	Ready to use enclosed in cobas <sup>®</sup> Liat <sup>®</sup> assay tube
Storage conditions	2-8°C
Shelf life	19 months
Performance	Superior sensitivity and specificity than other mPoC and Rapid PoC tests; equivalent to routine laboratory PCR
Limit of detection	Influenza A: $10^2$ - $10^1$ TCID <sub>50</sub> /mL.   Influenza B: $10^3$ - $10^1$ TCID <sub>50</sub> /mL.
Result	Influenza A: Detected or Not Detected   Influenza B: Detected or Not Detected
Time to result	~20 minutes

Source; cobas<sup>®</sup> Influenza A/B Product package insert

**For Roche Internal Use. Do Not distribute.**

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 23 | © 2016 Roche



# cobas<sup>®</sup> Influenza A/B

## Assay Design



Source: cobas<sup>®</sup> Influenza A/B Product package insert

**For Roche Internal Use. Do Not distribute.**

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 24 | © 2016 Roche





## Sample collection

For the testing of the cobas<sup>®</sup> Influenza A/B, the sample collection method is a nasopharyngeal swab. These are considered the most sensitive method for detecting upper respiratory viral infections<sup>1</sup>



Tilt the patient's head back at a 70° Angle

Insert the swab into the nostril straight back

Rotate the swab gently for 5-10 seconds

Remove swab and place swab into the transport media tube

Break the swab shaft and leave the swab inside the tube

UTM media with sample is ready for testing


### Key Points

Other sampling methods for Influenza testing are nasal swabs or nasal aspirates. Those have not been validated on the cobas<sup>®</sup> Liat<sup>®</sup> System.

If a sample in UTM media is not tested immediately, it can be stored at 2-8C for up to 72 hours.

UTM Tube

Nasopharyngeal swab

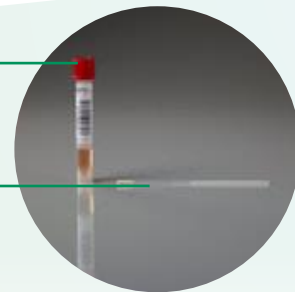

<http://jcm.asm.org/content/51/11/3880.long>  
<http://www.ncbi.nlm.nih.gov/pubmed/22723469>

For Roche Internal Use. Do Not distribute.

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 25 | © 2016 Roche

# cobas<sup>®</sup> Influenza A/B

## Strain coverage

### Influenza A seasonal H1

A/Brisbane/59/2007

A/New Caledonia/20/99

A/Solomon Island/3/2006

A/Mal/302/54

A/Denver/1/57

A/FM/1/47

A/PR/8/34

A/Weiss/43

### Influenza A seasonal H3

A/Brisbane/10/2007

A/Alice

A/MRC2

A/Hong Kong/8/68

A/Victoria/3/75

A/Wisconsin/67/05

A/Port Chalmers/1/73

A/Aichi/2/68

### Influenza A 2009 H1N1

A/NY/01/2009

A/NY/02/2009

A/NY/03/2009

A/New Jersey/8/76

A/Swine/1976/31

A/Swine/Iowa/15/30

### Influenza B

B/Florida/04/06

B/Malaysia/2506/04

B/Florida/7/04

B/Allen/45

B/GL/1739/54

B/Taiwan/2/62

B/Maryland/1/59

B/Mass/3/66

B/HongKong/5/72

B/Lee/40

The cobas<sup>®</sup> Influenza A/B Assay was evaluated with 22 Influenza A strains and 10 Influenza B strains.

Source: cobas<sup>®</sup> Influenza A/B Product package insert

For Roche Internal Use. Do Not distribute.

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 26 | © 2016 Roche



# cobas<sup>®</sup> Influenza A/B

## Clinical performance compared to culture

### Influenza A

#### Viral Culture

		Positive	Negative	Total
cobas <sup>®</sup> Liat <sup>®</sup> System	Positive	77	15 <sup>a</sup>	92
	Negative	2 <sup>b</sup>	690	692
	Total	79	705	784

	%	95% CI
Sensitivity	97.5%	(91.2% - 99.3%)
Specificity	97.9%	(96.5% - 98.7%)

**Key Point:** The sensitivity of the cobas<sup>®</sup> Influenza A/B exhibits excellent performance when compared to viral culture in a prospective study of 784 patients.

### Influenza B

#### Viral Culture

		Positive	Negative	Total
cobas <sup>®</sup> Liat <sup>®</sup> System	Positive	31	16 <sup>c</sup>	47
	Negative	1 <sup>d</sup>	736	737
	Total	32	752	784

	%	95% CI
Sensitivity	96.9%	(84.3% - 99.4%)
Specificity	97.9%	(96.6% - 98.7%)

<sup>a</sup> Of 15 cobas<sup>®</sup> Liat positive, Culture negative specimens, 9 were positive and 6 were negative by PCR/sequencing;

<sup>b</sup> 2 cobas<sup>®</sup> Liat negative, Culture positive specimens were positive by PCR/sequencing;

<sup>c</sup> Of 16 cobas<sup>®</sup> Liat positive, Culture negative specimens, 14 were positive and 2 were negative by PCR/sequencing;

<sup>d</sup> 1 cobas<sup>®</sup> Liat negative, Culture positive specimens was positive by PCR/sequencing and was negative by lab-based RT-PCR.

.Source: cobas<sup>®</sup> Influenza A/B Product package insert

**For Roche Internal Use. Do Not distribute.**

The cobas<sup>®</sup> Liat<sup>®</sup> System is not commercially available in all markets and some associated assays are currently in development. | page 27 | © 2016 Roche



# Comparison of Alere and Liat with the FilmArray test for detection of Flu A/B

Comparison of Alere and Liat for influenza A virus

Alere or Liat result	No. with FilmArray result:		
	Positive	Equivocal	Negative
Alere			
Positive	56	1	0
Negative	21	2	49
Liat			
Positive	77	3	0
Negative	0	0	49


Comparison of Alere and Liat for influenza B virus

Alere or Liat result	No. with FilmArray result:	
	Positive	Negative
Alere		
Positive	15	0
Negative	1	113
Liat		
Positive	16	0
Negative	0	113

J Clin Microbiol. 2016 Nov; 54(11): 2763–2766.



# Frequency distribution of cycle threshold (CT) values for samples positive for influenza A virus

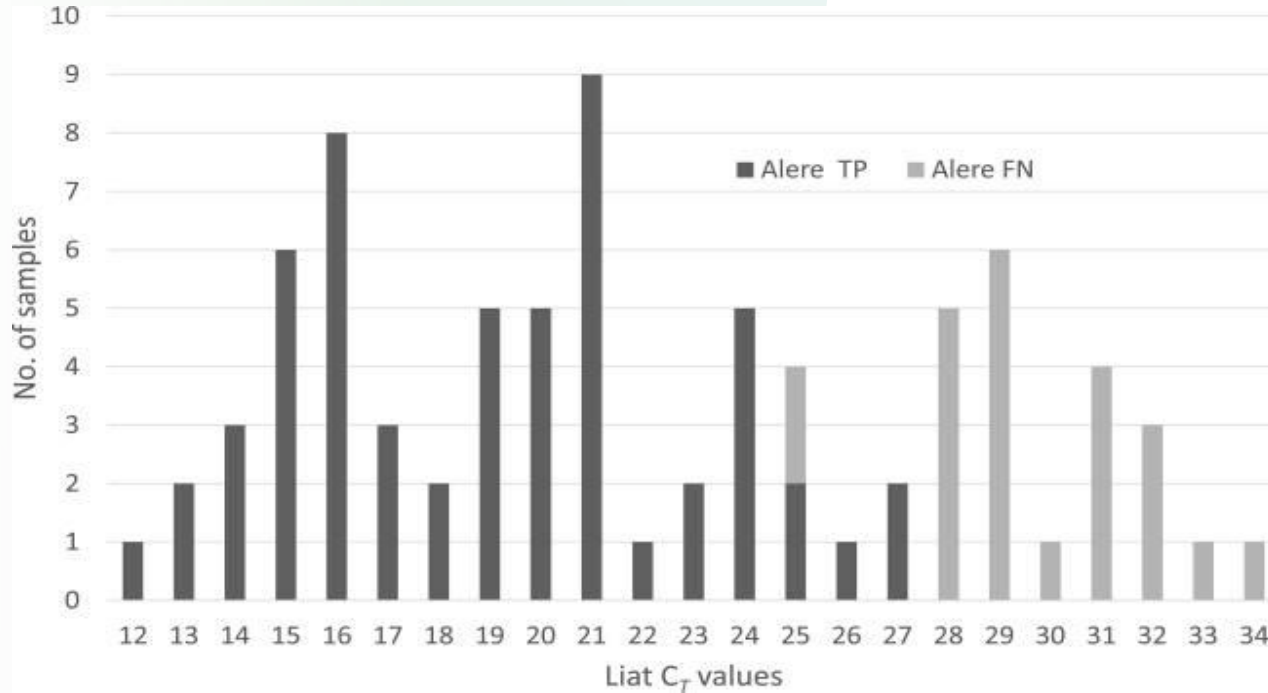

J Clin Microbiol. 2016 Nov; 54(11): 2763–2766.

# Multi-center evaluation of the cobas® Liat® Influenza A/B & RSV assay for rapid point of care diagnosis

**Table 1**  
Point-of-care testing study site characteristics.

Site Location	Site Classification	Number of Samples Tested	Number of CLIA-Waived Test Operators
Virginia	Clinic	322	2N, 3 MA, 1 RA
Connecticut	Clinic	18	1N, 1 MA
South Florida Site 1	Clinic	149	1N, 1 MA
South Florida Site 2	Clinic	90	2 MA
Nebraska	Clinic	130	5 MA
Texas	Clinic	20	3N
Indiana	Clinic	56	1N, 2 MA
California	Clinic	138	2 MA
Ohio	Emergency Department	148	1N, 1 MA
Massachusetts	Emergency Department	243	1N 2 RA
New York	Emergency Department	295	6 RA
Central Florida	Emergency Department	47	1 NS, 1 RA

N-Nurse, MA-Medical Assistant, RA-Research Assistant/Study Coordinator, NS-Nursing Student.

The Virginia, California, and Nebraska sites tested both prospective and retrospectively included samples. All other sites tested prospective samples only.

**Table 2**  
Patient demographics for prospectively collected NP swabs.

Age	Number of Prospective Samples	Percentage
≤ 5 years	320	23.5
6–21 years	408	30.0
22–59 years	505	37.1
≥ 60 years	128	9.4
<b>Total</b>	<b>1361</b>	
<b>Sex</b>		
Male	663	48.7
Female	698	51.3

# Sensitivity & Specificity of cobas Liat Assay

## Multi-Center Evaluation

Sensitivity and specificity of the cobas<sup>®</sup> Liat<sup>®</sup> Influenza A/B and RSV assay compared to ProFlu+ (reference test) for prospectively collected fresh and retrospectively included frozen NP Swabs (All Samples).

Influenza A		Reference Test		Total		%	95% CI
		Positive	Negative				
Liat	Positive	267	34 <sup>a</sup>	301	Sensitivity	99.6%	(97.9%–99.9%)
	Negative	1 <sup>b</sup>	1352	1353	Specificity	97.5%	(96.6%–98.3%)
	Total	268	1386	1654 <sup>c</sup>			

Influenza B		Reference Test		Total		%	95% CI
		Positive	Negative				
Liat	Positive	144	4 <sup>d</sup>	148	Sensitivity	99.3%	(96.2%–99.9%)
	Negative	1 <sup>e</sup>	1507	1508	Specificity	99.7%	(99.3%–99.9%)
	Total	145	1511	1656			

RSV		Reference Test		Total		%	95% CI
		Positive	Negative				
Liat	Positive	184	18 <sup>f</sup>	202	Sensitivity	96.8%	(93.3%–98.8%)
	Negative	6 <sup>g</sup>	1438	1444	Specificity	98.8%	(98.1%–99.3%)
	Total	190	1456	1646 <sup>h</sup>			

# Sequencing analysis of discordant samples results

## *Multi-Center Evaluation*

**Table 7**  
Sequencing analysis of cobas® Liat® Influenza A/B and RSV assay positive and negative discordant samples following comparison to ProFlu+ (reference test) results.

	cobas® Liat® Positive ProFlu+ Positive	cobas® Liat® Positive ProFlu+ Negative	cobas® Liat® Negative ProFlu+ Positive	cobas® Liat® Negative ProFlu+ Negative
Influenza A	267	34	1	1352
Influenza A Sequencing Results	N/A	Positive: 15/34 Negative: 19/34 Indeterminate: 2/34	Negative: 1/1	N/A
Influenza B	144	4	1	1507
Influenza B Sequencing Results	N/A	Positive: 3/4 Negative: 1/4	Negative: 1/1	N/A
RSV	184	18	6	1438
RSV Sequencing Results	N/A	Positive: 7/18 Negative: 11/18	Positive: 1 Negative: 5	N/A



# Population with high risk of getting serious complications from the flu

Adults  $\geq 65$

Children  $\leq 5$

Pregnant women

Immunocompromised

Chronic diseases

# Ideas & Conclusion

1. Need to focus on niche segment in hospital
  - ER
  - Specific Department in OPD
  - Hospitalized patients
2. Prevalence matters, cobas Liat can provide better PPV & NPV for the target population, provide better clinical value to the one who needs
3. In high prevalent scenario, the medical value is greater for cobas Liat Flu A/B system
4. The possibility for other assays?



For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 34 | © 2016 Roche



Roche Molecular Systems, Inc.  
4300 Hacienda Drive  
Pleasanton, CA USA

COBAS and LIAT are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

© 2016 Roche.

*This presentation is our intellectual property.  
Without our written consent, it shall neither be  
copied in any manner, nor used for  
manufacturing, nor communicated to third parties.*



For Roche Internal Use. Do Not distribute.

The cobas® Liat® System is not commercially available in all markets and some associated assays are currently in development. | page 35 | © 2016 Roche



# Technical specifications

## Dimensions

Height	19.0 cm (7.5")
Width	11.4 cm (4.5")
Length	24.1 cm (9.5")
Weight	3.76 kg (8.3 lbs.)

## Power

Input	100-240V AC /50-60Hz
Output	15V DC / 8.6A
Consumption	130 W on AC mains power
Safety class	II

## Ports

USB	2 Universal Serial Bus (USB) (maximum load of 250 mA)
Ethernet	RJ-45, TCP/IPUSB



## Temperature

Operating	15°C to 32°C (59 -90°F)
Storage	-12°C to +50°C (10-122°F)

## Humidity

Operating	15% - 80% (non-condensing)
Storage	5% - 90% (non-condensing)

## Altitude

2,000 m (6,500 ft.) above sea level

## Result Storage

4GB SD card

## Memory

256MB RAM / 512MB NAND Flash

## Barcode reader

Class 2 Laser Product

